
thermoflexi

miniland

B A B Y

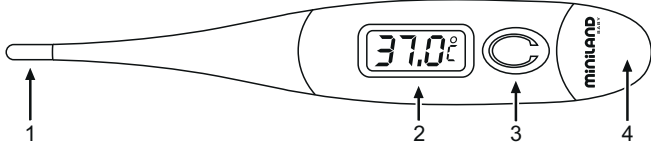ESPAÑOL

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ENGLISH

■ **1. DESCRIPTION OF THE PARTS OF THE THERMOMETER.**

- Probe
- LCD
- Power Button
- Battery Cover

■ **2. SPECIFICATIONS**

- Measurement Range - from 32.0°C to 42.9°C.
- Accuracy

Accuracy	Measurement range	Standard room temperature 22°C
±0.1°C	from 35.0°C to 42.0°C	
±0.2°C	remaining range	

2.3 Display unit °C (factory setting only)
2.4 Battery 1 button battery, 1.5V, LR41 type or similar
2.5 Battery life approx. 250 hours of continuous use
2.6 Shelf life: 3 years
2.7 Storage temperature -10°C to 40°C, humidity: 30% to 90%
2.8 Operation temperature 10°C to 35°C, humidity: 40% to 80%
2.9 Measuring time: between 10 seconds and 5 minutes
2.10 Weight with battery: approx. 12g

■ **3. HOW TO OPERATE**

- 1 Ensure that the thermometer has been turned off before taking temperature.
- 2 Press the power button. A short "Bf" signal will be heard, all the symbols will be turned on for one second to ensure all parts of the LCD is functioning properly, and then the last measurement data will be shown on the display for about 2 seconds. Finally, the flashing symbol °C will appear to indicate that the thermometer is ready to measure the body temperature.
- 3 During the measurement the LCD will display "Lo °C" if the measured temperature is below 32.0°C, or will display "Hi°C" if the measured temperature is above 43.0°C.
- 4 When the probe tip reaches its thermal equilibrium or peak, °C symbol stops flashing and the beeper will sound 10 times. The temperature is now ready to read and will not change when the thermometer is removed.
- 5 If the measured temperature is higher than 37.8°C, the fever warning function will make the beeper sound fleetly for about 10 seconds.
- 6 After measuring, turn off the unit by pressing the power button. If the power is left on, the "automatic shut-off function" will activate in approx. 9 minutes to save power. In continuous measurement, start each operation again from the beginning.

- The normal temperatures are different depending on the human body and measuring time.
- Wait at least 3 minutes between measurements.
- Measurement may be inaccurate when the battery is low.
- To get stabilize temperature, it is advice to measure for at least 3 minutes.
- Children should be under close supervision at all times while using this product. After monitoring the temperature, the unit should be removed from the child immediately.

■ **4. HOW TO REPLACE THE BATTERY**

When the battery flag ■ appears and blinks, the battery needs to be replaced.

- 1 Remove the battery cover by pulling it out.
- 2 Remove battery with a non-metal probe.
- 3 Replace with an alkaline LR41 battery or similar.
- 4 Make sure the positive side (+) facing up.
- 5 Close the cover securely.
- 6 Carefully discard the used battery and keep it out of children's reach.

■ **5. METHODS OF USE**

When measuring your body temperature, always measure it at the same place of your body and at the same time.
Whichever method is used, it is recommended that for 15 minutes prior to use, avoid eating or drinking any liquids, exercising, taking showers or baths, or smoking.

RECTAL USE

Commonly used for babies, young children, or when it is difficult to take an Oral or Axillary temperature.

1. Cover the thermometer's tip with a probe cover and lubricate with a water-soluble jelly for easier insertion. Do not use a petroleum jelly.
2. Gently insert the tip of the thermometer NO MORE THAN 1/2 inch into the rectum. If you detect resistance, hold the thermometer in place during measurement.
3. When the peak temperature is reached (usually 10-15 seconds), 10 beep signals will sound and the °C sign will stop flashing. The temperature is now ready to read and will not change when the thermometer is removed.

Remember that this method produces a temperature about 0.5-1°C higher than a temperature taken orally.

ORAL USE

1. Cover the thermometer's tip with a probe cover. The mouth should remain closed during the measurement to ensure an accurate reading. Be careful not to bite the thermometer.
3. When the peak temperature is reached (15-25 seconds), 10 beep signals will sound and the °C sign will stop flashing. The temperature is now ready to read and will not change when the thermometer is removed.

AXILLARY USE

This method used for babies and young children but less accurate and takes longer.

1. Wipe the underarm with a towel.
2. Cover the thermometer's tip with a probe cover. Place the probe tip well under the underarm. When the peak temperature is reached (usually 30 seconds but sometimes longer for this method), 10 beep signals will sound and the °C sign will stop flashing. The temperature is now ready to read and will not change when the thermometer is removed.
3. When the measurement is reached, the temperature °C lower than a temperature taken orally.
3. To continuously measure temperature 10 minutes and ignore the beep sound will get the temperature close to oral temperature.

■ **6. HOW TO CLEAN**

- 6.1 Clean the thermometer probe before and after use by wiping it with medical alcohol (70% isopropyl alcohol).
- 6.2 When taking continuous measurements, we recommend using a probe cover, which should not to be reused.
- 6.3 Cleaning is required before every measurement.

Nota: The product doesn't need any calibration and its use is recommended for 3 years.

■ **7. WARNING**

- 7.1 Swallowing a battery may be fatal.
- 2 Do not charge the battery.
- 3 Do not depose the battery in fire as it may explode.
- 4 Do not store the unit in direct sunlight or at a high temperature.
- 5 Do not bend or bite or pull the unit.
- 6 Do not attempt to disassemble the unit except to replace the battery.
- 7 The measurement reading may/be incorrect when the body temperature is lower than ambient temperature.
- 8 Suitable for measuring children's body temperature.
- 9 It is designed and manufactured complying with relevant requirements in ISO80601-2-56: Performance of Compact electrical thermometers (non-predictive and predictive) with maximum device.
- 10 It has passed EMC emission test, immunity test according to EN60601-1-2 and biocompatibility test according to MDD annex 1.
- 11 According to the degree of protection against electric shock, it is classified to BF, in accordance with EN60601-1.
- 12 This product meets the basic safety and essential performance requirements indicated in the IP22 conditioning test (IP22: Protection level against solid foreign objects and dripping water)
- 13 Avoid dropping the device or subjecting it to heavy knocks.
- 14 Never allow chemical solutions to come into contact with the thermometer and keep away from direct sunlight or heat sources as well as strong electromagnetic sources to avoid interference.
- 15 Dispose of the product and battery according to relevant national regulations.
- 16 Please take out the battery if the unit is going to be used for a long time.

Caution! Be sure that both the battery and the thermometer are kept away from children.

■ **8. BATTERY & PRODUCT DISPOSAL INFORMATION**

When the time comes to eliminate this product, please consider the environmental impact and take it to recognized recycling facility instead of disposing it with general household waste. Take the equipment to a waste disposal site. Plastic and metal parts that are used in the construction of this appliance can be separated into pure grade which allow recycling. Ask to your service center for details. Everyone of us can participate on the environmental protection.

NORMALIZED SYMBOLS

	Attention: Read the operating instructions for use!		Batch code
	Applied part of type BF		The name and the address of the manufacturer
	Disposal in accordance with Directive 2012/19/UE (WEEE)		Authorized Representative in the European Community
	Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Notified body 0434: Det Norske Veritas Certification AS		

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